

MEDICATOR[®]

Aerosol Maximizer

Convenience Kit

Part Number MM-802

Components:


- Corrugated Tube: 8"-24"
- Expiratory Port Plug
- Breathing Tube Adaptor

Healthline[®]
Aerosol Medicine

4610 Littlejohn Street • Baldwin Park, CA 91706
phone (626) 851-9616 • fax (626) 960-8700
www.Aerosol-Medicine.com

Principles of Operation

With certain patients it can become necessary to deliver aerosol medication through an extensible breathing tube to a mask or a mouthpiece. This requires moving the “expiratory port” from the Medicator manifold to the patient end of the breathing tube in order to eliminate rebreathed volume. The adaptor and expiratory port plug serve this purpose. The 8-24” breathing tube does not create a substantial barrier to aerosol delivery as it is essentially the same as that when the patient is directly attached to the Medicator manifold.

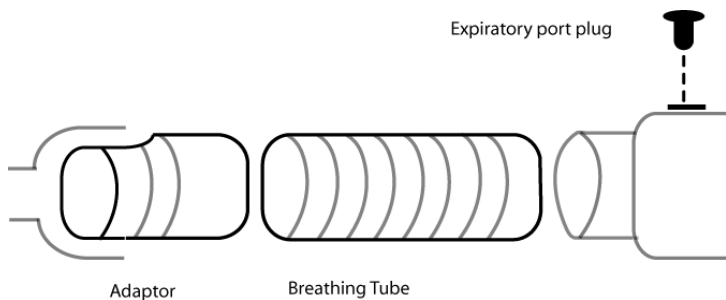

Directions for Use

1. Securely insert expiratory port plug into expiratory port on Medicator manifold.
2. Connect end of breathing tube to breathing port on Medicator manifold.
3. Attach patient mouthpiece (or optional facemask) to adaptor end of breathing tube.
4. Expand breathing tube to desired length for patient accommodation.
5. Begin prescribed treatment.